SOPIRA® Ject SOPIRA® Ject vario

- DE Gebrauchsanweisung
- GB Instructions for use
- FR Mode d'emploi
- ES Instrucciones de uso
- IT Istruzioni per l'uso
- NO Bruksanvisning

99000591/01 – DOG/12.09

Heraeus Kulzer GmbH
Grüner Weg 11
63450 Hanau (Germany)
info.dent@heraeus.com
www.heraeus-dental.com

Made in Germany

SOPIRA® Ject und SOPIRA® Ject vario

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Besondere Beachtung

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Instructions for use GB

Please note:

Due to US Medical Device Legislation and specific requirements for devices marketed in the USA the wording of the user instructions for the US may slightly differ from the standard English version.

Special observance

Please read the following information and application notes carefully.

The product may only be used by experienced medical staff in accordance with these instructions. Heraeus does not give any recommendation regarding a method of treatment. The treating medical specialist staff is responsible for the manner of application and for the selection of the patient.

The non-observance or contravention of the instructions will lead to the expiry of the guarantee and to endangering the safety of the patient.

If used in combination with further products, the instructions for use and the compatibility statements of those products shall always be taken into account. Please check the product for completeness and intactness before application.

If substantiated cause for doubts regarding the completeness and the intactness are on hand, then do not use the product.

Product specification/compatibility

The SOPIRA Ject is available in two various designs. The SOPIRA Ject with swing-out mechanism to open the cartridge holder (the cartridge holder is firmly connected to the syringe), and the SOPIRA Ject vario with slip-on device for separate cartridge holders (the cartridge holders can be replaced). Both models offer self aspiration as well as manual aspiration by means of hooklets.

Indications

Infiltration- and nerve block anaesthesia in dentistry

Contraindications

The contraindications to be observed are those of the applied anaesthetic.

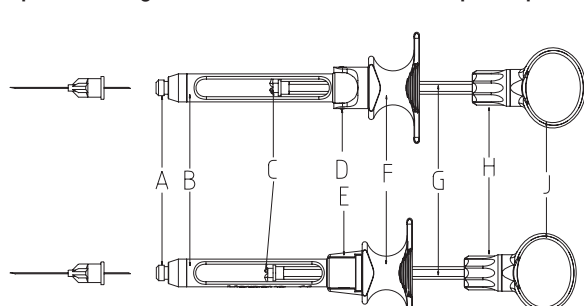
Complications

No complications are known to occur under professional, proper application.

Warning indications

The aspiration hooklets (C) have to be in retracted position when inserting or removing the cartridges. They may otherwise be bent. Extend hooklets only to connect cartridge/plunger, please retract again when separating the connection! Retract the hooklets by turning the knurled screw (H) to the left hand side until the limiting stop becomes effective!

Special warning indications for use with other compatible products:

- A** Screw thread for cannula.
- B** Cartridge holder with viewing windows.
- C** Three pivatable (swing-in and -out) aspiration hooklets at the end of the plunger provide for stable anchoring in the hollow stopper of the cartridge, and they ensure for smooth guidance when injecting and aspirating.
- D** Joint with swing-out mechanism (for opening the cartridge holder).
- E** Slip-on device for mounting the cartridge holder.
- F** Finger rest.
- G** Plunger.
- H** Knurled screw to extend and retract the hooklets. The hooklets are extended by turning the knurled screw (H) to the right-hand side (clockwise), and turning left (counter clockwise) will retract them. The screw – mechanism is equipped with a locking device. It is engaged as soon as the hooklets have been completely extended by turning to the right-hand side. An unintentional separation of the connection is impossible. The effect of the locking device may be noticed as a slight initial resistance when turning the knurled screw to the left-hand side to separate the connection.
- J** Thumb plate ring.

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Stand: 12.2009

Sequence of use

1. Insert the cartridge
 - a) *SOPIRA Ject:*
Pull the plunger (G) back completely. A spring resistance has to be overcome towards the end. Swing cartridge holder (B) out at a right angle. The holder is in open position. Insert the cartridge. Swing the holder back in alignment with the longitudinal axis. Thereby, completely retract the plunger backwards against the resistance of the spring. Release the plunger against the spring resistance. The cartridge will slip into the correct functioning position, centred in the cartridge holder by a guide ring.
 - b) *SOPIRA Ject vario:*
Pull the plunger (G) back completely. A spring resistance has to be overcome towards the end. Remove the cartridge holder (B) from the syringe by pulling it out of its rest laterally. Insert the cartridge into the cartridge holder. Slip the cartridge holder with the inserted cartridge into the rest of the syringe body. Thereby, completely retract the plunger backwards against the resistance of the spring. Release the plunger against the spring resistance. The cartridge will slip into the correct functioning position, centred in the cartridge holder by a guide ring.
2. The plunger (G) can be advanced forward several millimetres against the inserted cartridge: The front end of the plunger with the hooklets (C) will be fitted tightly into the recessed void of the hollow stopper. The connection is created by turning the knurled screw (H) to the hooklets have been completely extended.
3. The cannula is tightly screwed onto the cartridge holder.
4. Aspiration
The aspiration device is essentially based on the creation of a stable connection between the hollow stopper of the cartridge and the plunger of the syringe, which will bear tractive force and force of pressure. This jam-free fastened connection permits the performance of aspiration and injection without problems. General complications relating to local anaesthesia during dental treatment are often the result of an unintentional intravenous or intra-arterial injection. Failures of onset may also occur in case of an inadvertent intravascular injection, since the local anaesthetic is carried away quickly from the location of the application.
The aspiration check, performed "lege artis", will largely provide protection from injecting into a blood vessel.
5. The removal of the cartridge is performed in reverse order.

Assembly/dismantling

see "Application"

Cleaning/sterilisation

After use, the dental cartridge syringe is to be sterilised through autoclaving before further use. The syringe is to be cleaned by suitable means before sterilisation. If the free movement of the locking mechanism should perhaps be impaired despite regular cleaning, then it is recommendable to clean the syringe body in the ultrasonic cleaning bath.

Cleaning

The methods, facilities and equipment validated by the user are always to be observed and checked for compatibility with the details given here in every case.

Reusable instruments and equipment must be checked regarding functionality and intactness immediately before and after use, as well as after cleaning operations.

Contaminated instruments are to be processed as soon as possible after use; dismantle the dismountable instrument into its components according to the dismantling instructions.

Rinse the instrument with clear, flowing water before machine-cleaning and disinfection. Remove coarse contaminations carefully by means of a soft cloth or soft brush. Clean manually or mechanically. Strictly avoid unwashed areas/product residue patterns.

The following cleaning agents may be used: deconex, Borer Chemie (cleaner), mucapur-AF, Merz Consumer (cleaner), neodisher z dental, Dr. Weigert (cleaner, neutraliser).

Sterilising with steam

Steam sterilisation must be performed according to a validated steam sterilisation process (e.g. steriliser in accordance with DIN EN 285 and validated in accordance with DIN EN 17665-1). If sterilisation is to be performed using the fractionated vacuum method, then the 134°C / 3 bar – program is to be used, employing a minimum – hold time of 4 minutes Drying time: 20 min. Allow equipment/instruments to cool down to room temperature before renewed use. After steam sterilisation, store exclusively in suitable containers designated for such purpose.

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